
Scan this code with your smart phone for additional TRUEresult information. Always refer to the Owner's Booklet for detailed product information.

## Checking Your Blood Glucose Using TRUEresult®

*A guide for testing your blood glucose*

### Simple Steps to Test Your Blood Glucose

## Meter, Test Strips & System Components

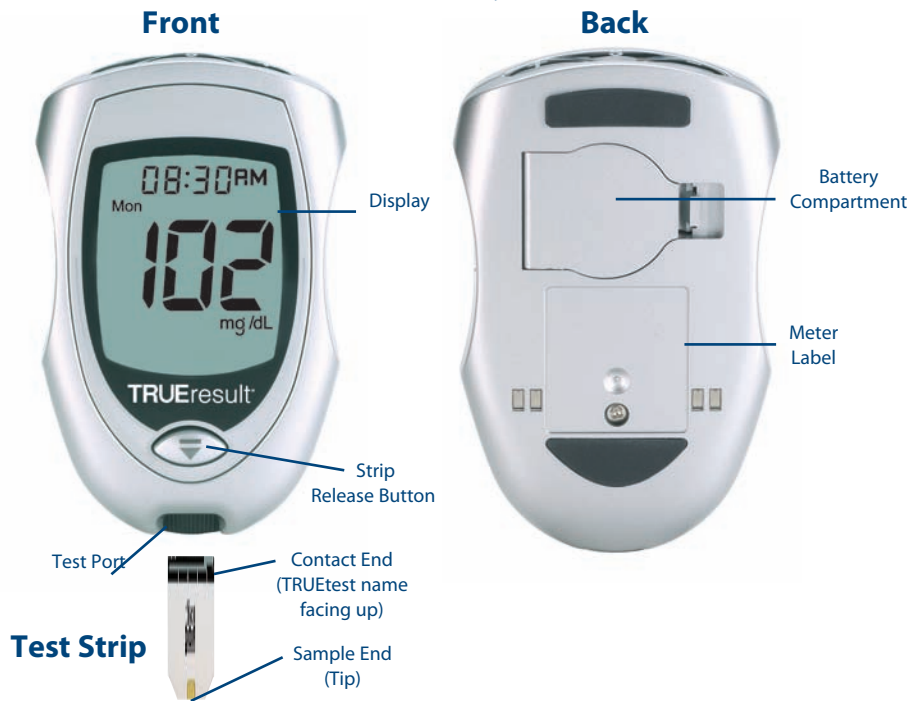### Blood Glucose Testing

**1**

Wash your hands with warm, soapy water.

**2**

Remove Test Strip from vial and insert into Test Port with Contact End (TRUEtest name facing up). The Meter turns on.

**3**

Lance your finger.

**4**

Allow a blood drop to form, touch Tip of Strip to top of blood drop and allow blood to be drawn into Strip. Remove Test Strip Sample Tip from sample drop immediately after Meter beeps and dashes appear across Meter Display.

**CAUTION!** Holding the Strip to the sample too long after Meter begins testing may cause inaccurate results.

**5**

In as little as 4 seconds, glucose result will be displayed.

**6**

Hold Meter Strip pointing down. Press Strip Release Button to discard the used Strip.

#### WARNING!

**NEVER** reuse Test Strips. **NEVER** wipe Test Strips with water, alcohol or any cleaner. **DO NOT** attempt to remove blood or control sample from Test Strips or clean Test Strips and re-use. Reuse of Test Strips will cause inaccurate results. **NEVER** add a second drop of sample to the Strip. Adding more sample gives an error message.

This guide provides a brief overview on testing your blood glucose. For more information on the TRUEresult System, see your Owner's Booklet. For assistance using your TRUEresult System, call our Customer Care at 1-800-803-6025.

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