

TRUEmanager™ 3.0.1 Release Notes

- The overall glucose target range in the system defaults is set to 100 to 130 mg/dL plasma when TRUEmanager™ is installed. The recommended target range from the ADA guidelines is 70 to 130 mg/dL plasma. The user is encouraged to set their target range defaults to the ADA guideline setting unless a physician recommends another default range.
- Windows Vista Installation - to install TRUEmanager™ on Windows the User Account Control Feature (UAC) needs to be temporarily disabled. To do this the user needs to go into the “User Accounts” section in the Control Panel and select “Turn User Account Control On or Off” and then de-select (turn off) the option and restart their computer.
- .NET 2.0 Service Pack 1 prerequisite fails installation – when attempting to install TRUEmanager™ on a .NET development machine that has the .NET 2.0 Service Pack 1.0 or later already installed .NET 2.0 may fail and the setup will indicate to try to reinstall .NET 2.0 before continuing. Installation can continue without doing this and will complete successfully.